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www.wtavet.com.br

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WTA - Guns

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To prolong the product usefulness please read and follow the instructions in this leaflet carefully.

Warning for important
operating instructions.

To additional information, please contact our Technical Assistance Department by e-mail or phone.

Opening hours: From Monday to Friday, 8:30 am to 5:30 pm (GMT -03:00)

Phone: + 1 (979) 324 - 6168

E-mail: usa1@wtatech.com

©WTA - Watanabe Tecnologia Aplicada Eireli - EPP - All rights reserved.

PRESENTATION

Congratulations on your choice. You have just purchased a WTA product, company Congratulations on your choice. You have just purchased a WTA product, a brand that adds quality and technology in what it manufactures.

WTA is a Brazilian company specialized in solutions for animal reproduction. Our products are developed for working with cattle, sheep and horses, with recognized approval by the most renowned specialists in the world.

PRODUCTS APPLICATION

The WTA guns are indicated for artificial insemination. They have quick and automatic sheath locking. Made of stainless steel with aluminum finish or AB. Easy to use, disassemble and clean. They provide practicality, safety and hygiene for professional work.

USE

Insert the sheath completely to the end - hear a "click". Before use check that the locking has been successful. Press the trigger to remove the sheath.

CLEANING

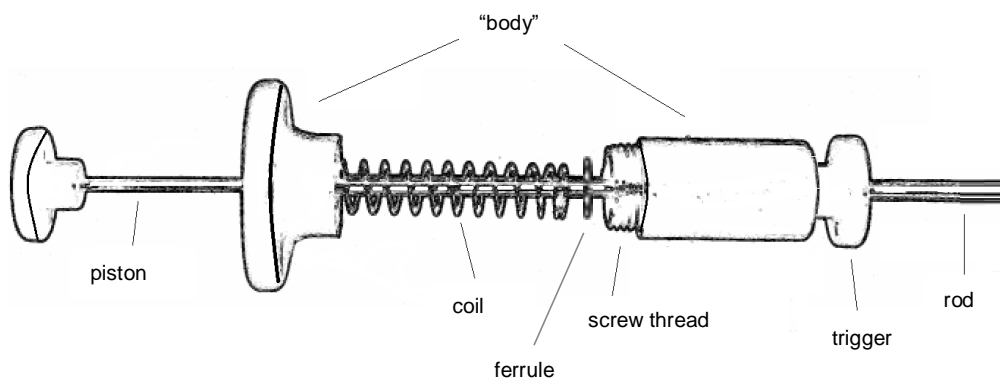
Use 70% ethanol or solution with water and neutral detergent for the hygiene of the plastic and metallic components. Do not use boiling water (may cause the rod warp or deform the body of the device).

Polymer remnants of the sheaths may accumulate inside the "body" difficult to lock the sheath. To clean the inside of the device, unscrew the rear section and remove any debris. (be careful not to damage the coil or lose the inner ferrule).

Proceed with the opening only when necessary to prevent premature wear of the threads.

Keep the device horizontal and avoid overlapping of heavy objects to preserve its characteristics.

WARRANTY

WTA - Watanabe Applied Technology Eireli - EPP follows strict internal quality control standards in the products manufacture. In addition to the legal guarantee of 90 (ninety) days, provided for in the Consumer Protection Code, it guarantees its customers an additional 90 (ninety) days. Warranty, counted from the date of issue of the purchase invoice, against the usual wear and provided that the device has been used in accordance with the guidelines contained in this leaflet.

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Images contained are merely illustrative.

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Todos os direitos reservados | All rights reserved