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User's Manual

www.wtavet.com.br

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Index

USER GUIDE	19
WARNING	19
Copyright	19
PRESENTATION.....	20
GENERAL DESCRIPTION.....	20
ACCESSORIES.....	20
TECHNICAL FEATURES.....	21
FUNCTION SPECIFICATION.....	22
IMPORTANT PRECAUTIONS	23
Repairs.....	23
Cares and Transport	23
Cleaning	23
Falling Objects Inside the Equipment	23
Power Cable.....	23
Battery.....	23
SETTING AND OPERATION	24
Charging the Battery.....	24
Insert the Battery.....	24
Turn on the Equipment.....	25
Temperature Adjusting	25
Date and Hour Adjusting	26
FUNCTIONALITY.....	26
LOW POWER MODE	27
ALARMS	28
BLUETOOTH	28
APPLICATIVE.....	28
WARRANTY.....	30
PROBLEM RESOLUTION	32
TECHNICAL ASSISTANCE SERVICE / CUSTOMER SERVICE	33

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USER GUIDE

WARNING

To prevent accidents and/or damage, carefully and completely read and follow this instruction guide before using the product.

Warning for dangerous voltage presence.

Warning for important operating instructions.

Copyright

Versatile Embryos/Oocytes Transporter WTA - User Guide
WTA, Cravinhos, State of São Paulo, Brazil.

The content of this guide is protected by copyright laws and cannot be reproduced, distributed, disclosed nor stored without prior written permission of WTA - Watanabe Tecnologia Aplicada Eireli - EPP.

In order to obtain authorization, additional information or clarification, please contact our Technical Assistance Department by e-mail, letter or phone.

Phone: +1 (979) 324-6168

e-mail: usa1@wtatech.com

Opening hours: From Monday to Friday, 8:30 am to 5:30 pm (GMT -03:00)

©WTA - Watanabe Tecnologia Aplicada Eireli - All rights reserved.

Comments

The information, illustrations and specifications contained in this user guide assume the latest features of the product available at the time of its publication.

Due to the use of new technologies and raw materials, WTA reserves the right to change the design and/or technical specifications of the product always as necessary and without prior notice, without incurring in any obligations.

This manual is integral part of the product. Therefore, the consumer should save it for any queries.

PRESENTATION

Congratulations on your choice. You have just purchased a WTA product, a brand that adds quality and technology in what it manufactures.

WTA is a Brazilian company specialized in solutions for animal reproduction. Our products are developed for working with cattle, sheep and horses, with recognized approval by the most renowned specialists in the world.

Its anodized aluminum frame offers, besides an elegant design, conditions that facilitate its internal and external cleaning.

For the best performance of the product we recommend reading this manual.

GENERAL DESCRIPTION


The WTA Versatile Transporter can be modified to carry Oocytes or Embryos according to your needs. The functionality Oocytes (equipment in vertical), allows to accommodate up to 56 WTA microtubes of 1.2 mL. The Embryo function (equipment horizontally) allows up to 198 small straws (0.250 mL) to be individually accommodated

Constructed in aluminum, it has a sophisticated electronic heating system with high precision sensors for greater temperature control. Its battery guarantees up to 80h of continuous operation (at ambient temperature of 25°C). It has Datalog of the last 24 hours of operation and alarms (audible and visual) for thermal oscillation. It also allows the monitoring Oocytes or Embryos condition parameters by Bluetooth.

These features offer, in addition to its practicality, safety to transport biological material from the field to the IVF laboratory and vice versa.

ACCESSORIES

Be sure that these accessories are intact and included in the package:

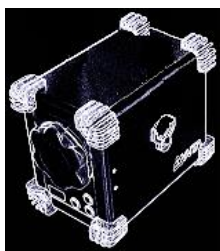Versatile
Transporter

Oocyte
Support Discs

Embryos
Cylinder

Battery and
Charger

Strap

1 Instruction Manual.


Note: If any accessory is missing or damaged, contact the WTA customer service department within 3 (three) days of the product delivery date.

TECHNICAL FEATURES

- Equipment manufactured in aluminum.
- Weight without block.: 7.72lbs.
 - Oocytes block: 0,90 lbs.
 - Embryos block: 3,03 lbs.
- Dimensions (LxWxH): 275 x 175 x 230mm.
- Capacity: - Oocytes: 56 microtubes of 1.2mL ($\varnothing = 14,2$ mm).
(Block with 4 thermal discs)
 - Embryos: 198 straws (0.250 mL) and 6 extra cavities to heat tips applicators.
- Temperature range: from 34,0 to 39,0°C. (93.2 to 102.2°F).
- Temperature variation on the display = 0.1 °C.
- Battery autonomy: 70 hours. (Test conditions: Ambient temperature 77° F; Block temperature 101.3° F).
- Audible temperature alarm
- Rechargeable Battery: 99,6Wh; 8850mAh; 11,25Vdc).
- Original battery specifications durability: Up to 80% of initial capacity after 800 cycles.
- Product warranty 3 years.


FUNCTION SPECIFICATION

Figure 1-A. Front and top view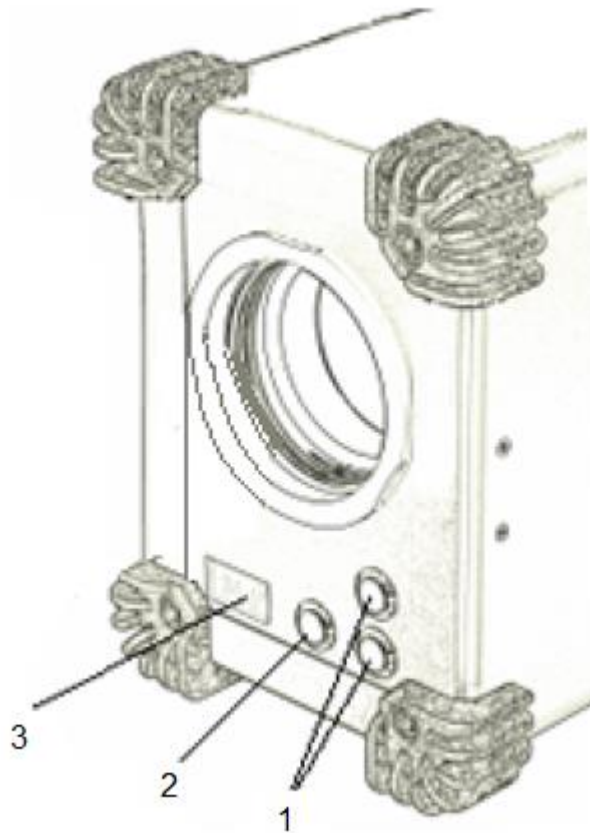

- 1- Adjustment keys “+” e “-”
- 2 - MENU
- 3 - Display
- 4 – ON/OFF switch

Figure 1-B. Rear section

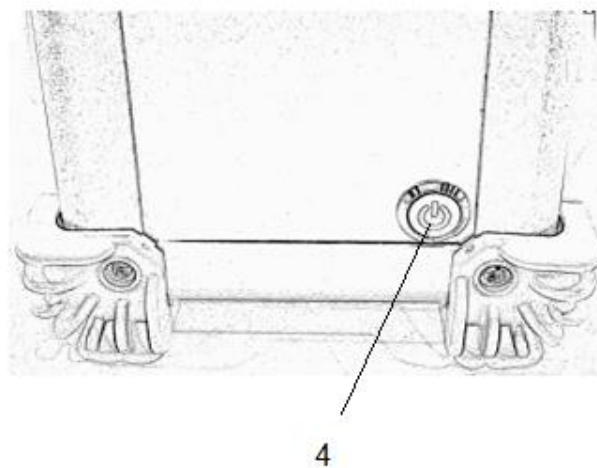IMPORTANT PRECAUTIONS

Repairs:

Do not open the device. All repairs and maintenance should be performed by authorized professionals or by WTA service.

Cares and Transport:

Avoid shaking during transport which can damage it. Make sure that the equipment is properly seated and protected against any accidents.

Cleaning:

Clean the exterior with a soft cloth slightly moistened with water. Do not use alcohol, thinner or other solvents as they will cause damage device's finishing.

Falling Objects Inside the Equipment:

Do not place recipients or cups containing liquid, or coins, paper clips or other metallic objects on the control module. If any liquid or any objects falls inside the equipment, turn off the power immediately and contact the service center.

Power Cable:

Do not step on the battery power cable or put any kind of objects (furnishing, etc.) over it.

Battery:

In case of replacement follow the original specifications. If in doubt, seek authorized technical assistance.

SETTING AND OPERATION

Before using the equipment, fully charge the battery. To guarantee the maximum battery lifespan up to 10 hours, make sure that it receives a full charge. The LED battery light on the charger allows you to monitor the "status" of the charge:

- Red and Green flashing: Recognition and initialization.
- Red light: Battery charging.
- Green light: Battery charged.
- Red flashing: Battery fault. Disconnect, wait a few minutes and reconnect charger.
- Red flashing in cycles of 7 times. Battery damaged.

Charging the Battery

Safety: insert the battery into your charger before connecting the power cord to the outlet.

The charge level can be checked directly on the battery by pressing the side button, if

- 4 lights = charge between 75 and 100%.
- 3 lights = charge between 50 and 75%.
- 2 lights = charge between 50% and 25%.
- 1 lights = charge between 5 and 25%.
- No lights illuminated = battery dead


Insert the Battery

Make sure the on / off button (4) is off.

This equipment works only with specific battery. Other electrical sources such as vehicle batteries or AC / DC converters are not possible

Using a metal wrench, remove the fixing screw and the acrylic cover located at the bottom. Then, insert the RRC2020 battery.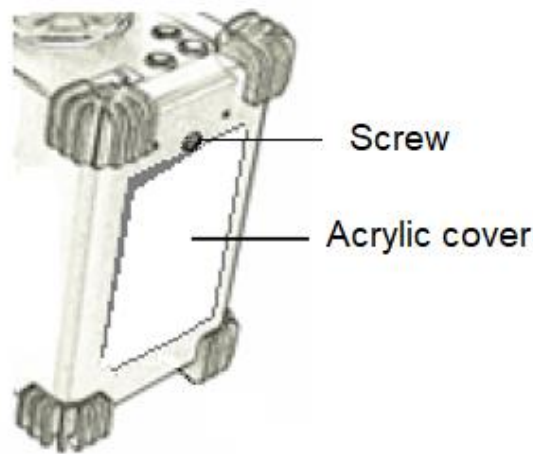

Replace the acrylic cover and secure the screw.

Turn on the Equipment

To turn on, press the On/Off button (4) located on the back and wait for the equipment to start up. The display (3) will show the WTA logo, serial number and firmware version.

Temperature Adjusting

After the equipment information is presented, the display will show Setpoint. At this time use the "+" and "-" keys (1) to adjust ideal temperature, between 34 to 39 °C. Wait a few seconds for a beep sound confirmation. After the adjustment is complete, heating will start.

Wait a few minutes for the oocyte or embryo storage block temperature stabilize.


This equipment will not provide cooling if temperature setted was lower than ambient temperature.

When the desired temperature is reached, a “beep” sound will be emitted.

Note: It is recommended to remove the battery if you are not going to use the equipment for a long time.

The display will show the instant temperature and the remaining charge percentage.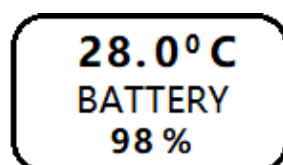

Date and Hour Adjusting

To adjust the time and date: Turn off the equipment, hold the Menu (2) and “+” (1) buttons simultaneously and press the ON/OFF button (4). Wait for the equipment to start the adjustment mode.

Use the “+” and “-“ (1) keys to adjust: seconds; minutes and hour. Then adjust: day, month and year respectively. Use the Menu key (2) to confirm each value and move on to the next one. At the end, use the Menu key (2) to exit the adjustment mode. Then the setpoint mode (temperature adjustment) will be enabled. If necessary, adjust the temperature using the “+” and “-“ keys (1). Otherwise, just wait a few seconds until the confirmation “beep” sounds. After the adjustment is complete, heating will start.


When the desired temperature is reached, a “beep” sound will be emitted.

FUNCTIONALITY

To switch between embryos or oocytes transporters, just change the heating block and reposition the equipment.


The straws with embryos must be placed individually in each well of the block (Figure 2 A).

Figure 2 - Arrangement according functionality.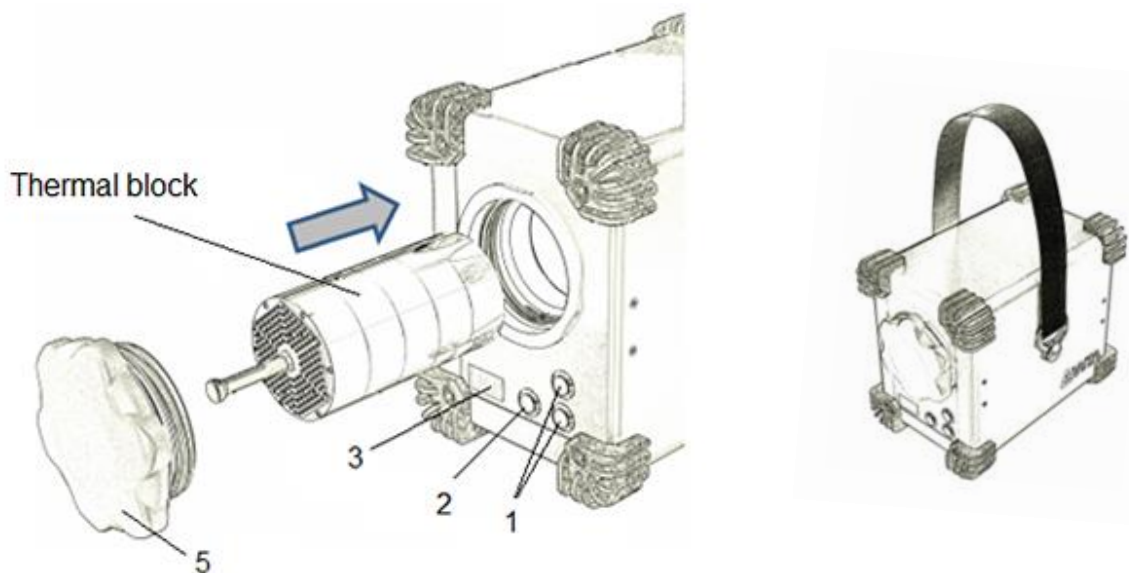

A - Embryos straws transport
(Horizontal)

The microtubes with oocytes must be placed in thermal discs, each of which accommodates 14 microtubes (Figure 2 B).

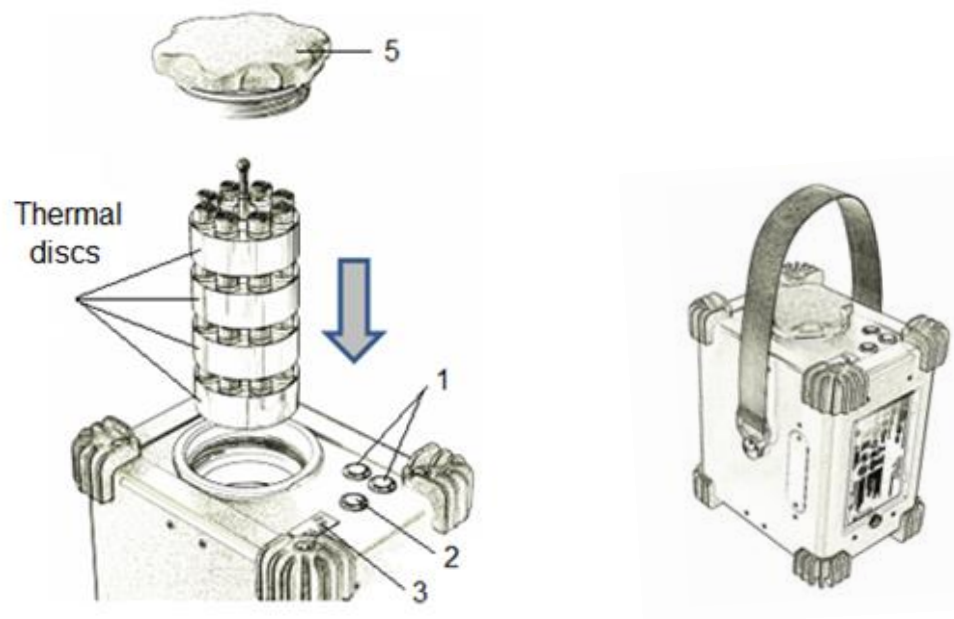B - Oocytes microtubes transport
(Vertical)

Once the temperature is stabilized, the biological material can be inserted into the block and this inside the equipment. At the end, make sure that the cover (5) is properly closed and adjust the handle.

Detailed block temperature records (every 5 minutes) are stored in the datalog and can be obtained by the applicative.

Note: When the battery charge reaches 2%, the device will automatically turn off.

LOW POWER MODE (Energy Saving)

To activate battery saving mode, press the Menu (2) and “+” (1) buttons simultaneously for 3 seconds. The display (3) and the audible alarm will be turned off. A confirmation beep will be emitted.

To exit low power mode, simultaneously press Menu (2) and “-“ (1) buttons for 3 seconds. A confirmation beep will be emitted and the display (3) and the audible alarm will be reactivated.

Note: During the energy saving mode, the on/off button (4) must be ON.

If the equipment is switched off during economy mode, when turning it back on, the Low Power function will be deactivated (display and audible alarm will be active) and the temperature will be resumed with the same value previously set.

Note: Enabling/disabling this mode can also be done by the applicative via “Bluetooth”.

ALARMS

in case of: (i) the block temperature exceeds 0.5° C to the setpoint or (ii) battery charge is low, the audible alarm will be activated and the message <High temperature> or <Battery below 5%> will be respectively shown on the display.

BLUETOOTH

To activate “Bluetooth” press the menu key (2) for 5 seconds. The “Bluetooth” symbol () will appear on display (upper left corner). Only a single device can connect to the Transporter. If the “Bluetooth” does not connect within 1 minute, the module will be disabled automatically.

APPLICATIVE


WTA provides a “Bluetooth” applicative (Figure 3) for devices that can be obtained by Google Play. Search for WTAtch => Install.

Figure 3 - WTAtch applicative

This applicative allows user to connect to the equipment (Fig. 4 - A1) to monitor in real time: block temperature; external temperature; battery charge; setpoint (A2) and access menu applicative (A3).

Figure 4 - Monitoring and Adjust Screens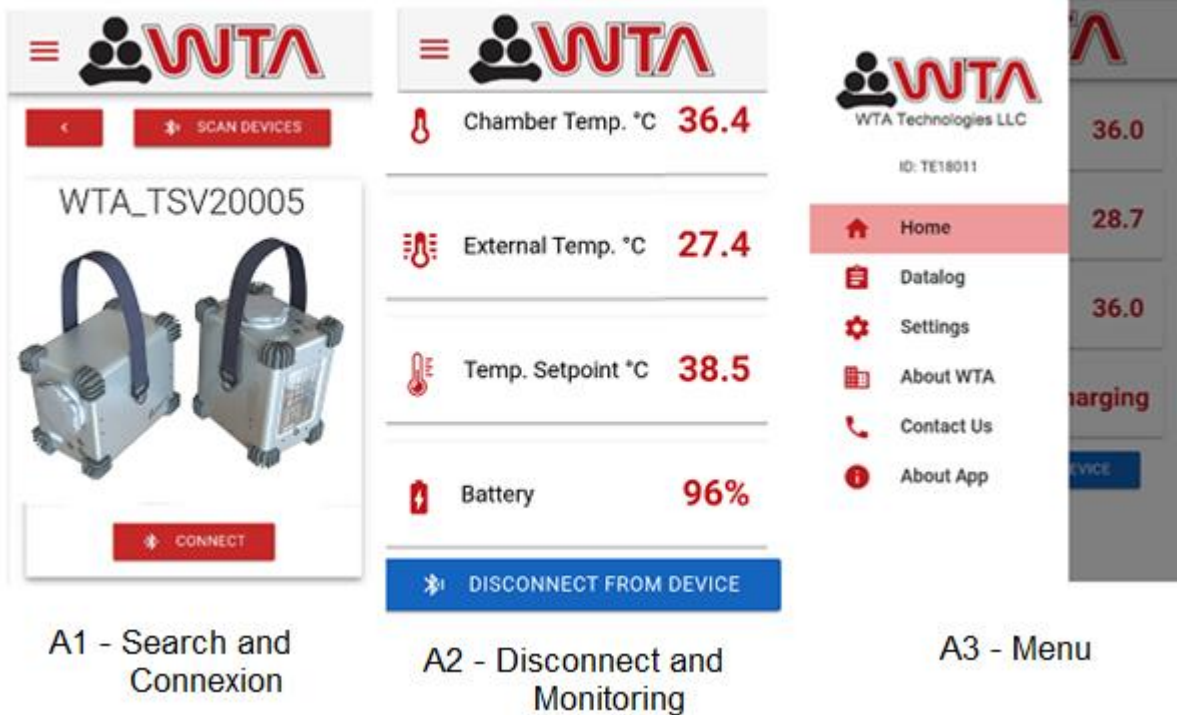


A1 - Search and Connexion

A2 - Disconnect and Monitoring

A3 - Menu

The last twenty-four hours information in datalog can be viewed (B1, B2 and B3), including saving it in PDF (B4).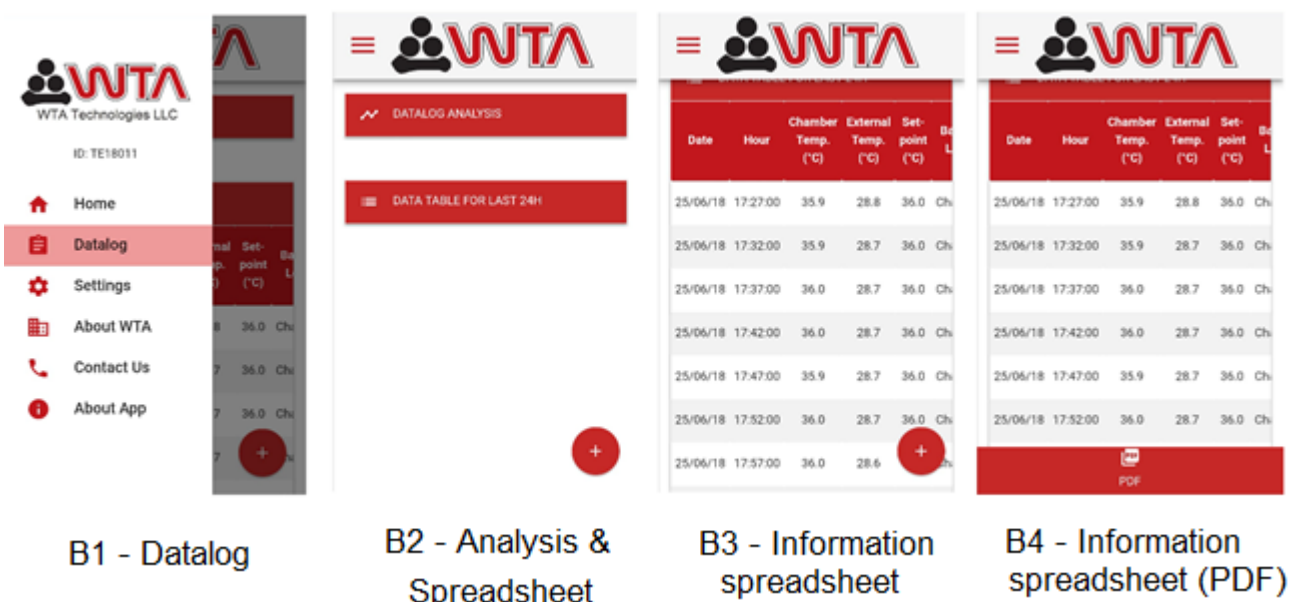


B1 - Datalog

B2 - Analysis & Spreadsheet

B3 - Information spreadsheet

B4 - Information spreadsheet (PDF)

It also allows making temperature settings and activating/deactivating the Low Power mode (C1 and C2).

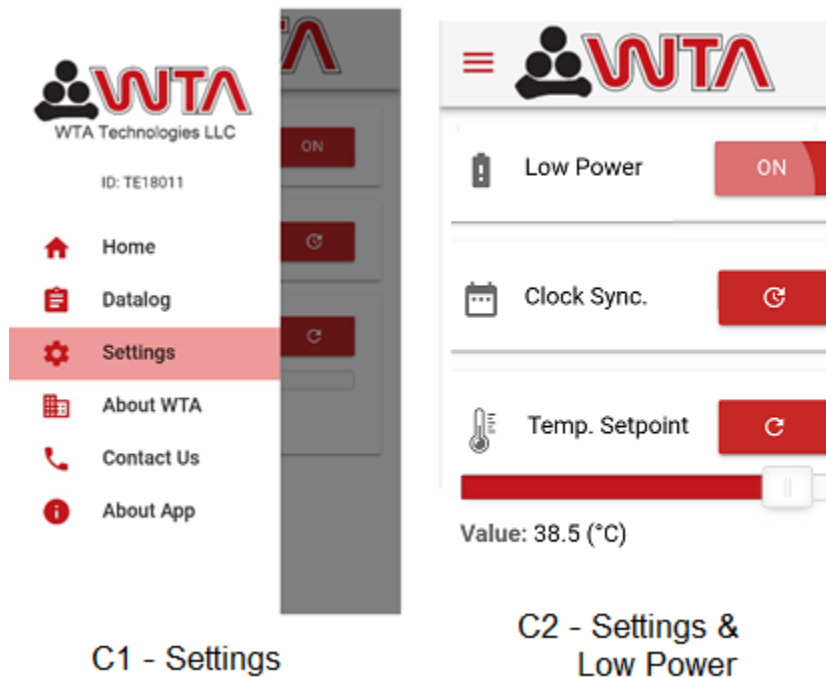WARRANTY

WTA - Watanabe Applied Technology Eireli - EPP follows strict internal quality control standards in the products manufacture. In addition to the legal guarantee of 90 (ninety) days, provided for in the Consumer Protection Code, it guarantees its customers an additional 33-month Warranty, counted from the date of issue of the purchase invoice, provided that the equipment has been installed and according to the guidelines contained in the User's Manual.

Even if ownership of the equipment is transferred, the warranty will continue to be valid according to the terms and conditions specified above.

This product has been carefully evaluated at every stage of the manufacturing process. This warranty will always accompany you, valid only in the country where the product was purchased.

It is understood by Additional Warranty, the free repair of the product and / or the replacement of parts that, according to technical opinion of the WTA, present a manufacturing defect.

Only WTA Technical Assistance professionals are entitled to repair defects for the guarantee, by presenting the sales tax (original) to the first buyer / consumer.

The Warranty Service

- Assistance in WTA technical assistance.
- Parts used for maintenance, subject to the exceptions provided in the item "Exclusion of Warranty".

The product should be sent directly to the WTA observing that:

- The product must be properly packed, if possible in its original packaging, together with the manual, cables and other accessories that have been delivered at the time of purchase.
- The product will be pre-evaluated by WTA technicians prior to repair.
- Shipping, freight and packaging costs will only be charged to WTA during the legal guarantee period (90 days). After this period such costs will be the customer's responsibility.

Voiding Warranty

This warranty does not cover defects or defects arising from the following situations:

- Installation and use of the product in an improper way or in disagreement with the respective guidelines and information described in this Manual, including installation, external adjustment, cleaning and use of accessories not in accordance with technical specifications.
- Defects caused by accidents, falls, atmospheric phenomena (rains, floods, electric discharges), excessive humidity (water, liquids in general), salinity, sinister (theft or theft), loss or improper transport of the product.
- Installation, adjustment, repair or servicing by persons outside WTA service.
- Invoice and / or serial number adulterated or withdrawn.
- Modifications, alterations or violation of the original product.
- Product connected to the mains improper or subject to excessive fluctuations.
- The product is purchased in balance, showcase or demonstration, this being characterized by means of identification in the invoice of purchase and / or price lower than the one practiced in the market.

Important: WTA is not liable for damage to equipment during transportation. When purchasing your product make sure on the delivery act that it is not defective or with apparent defects. If you notice any irregularities, return the product immediately.

This Warranty Statement Does Not Include

- Periodic reviews.
- Displacement and expense of technician by authorized WTA service.

Responsibility

The WTA has no control or influence over the conditions under which the equipment is used, as well as the methodologies employed. Thus, WTA assumes no responsibility for the results, use and / or performance of the equipment (including: damages for loss of profits, interruption of business activity, loss of business information or other pecuniary losses). WTA expects that the equipment will be used by trained personnel. The purchaser acknowledges that the equipment is subject to all terms and conditions described in this manual.

PROBLEM RESOLUTION

Occasionally many apparent defects may be something simple to solve. If you notice any problems with the equipment, check the following procedures.

Equipment Does Not Turn On

- Check if machine is turned on, the button ON/OFF must be pressed.
- Check that the equipment is in the Low Power mode. If so, disable the function.
- Check the battery charge level; it may be low. Remove and insert it into the charger.

Red light: The battery is charging.

Green light: The battery is charged and can be use.

Equipment Not Heating

- Check that the temperature is set correctly.

The Equipment Does Not Operate on the Battery

- See item "*Equipment Does Not Turn On*" above.

The Battery Charger LED Does Not Illuminate

- Check the outlet is energized by connecting other equipment I the same outlet.

If the problem persists, contact your service representative.

TECHNICAL ASSISTANCE SERVICE / CUSTOMER SERVICE

If there is any doubt or if it is necessary additional information about warranty, assembly, installation, adjustment or utilization of the product, please contact our Customer Service Department:

Opening hours: From Monday to Friday, 8:30 am to 5:30 pm (GMT -03:00)

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Images contained in the Guide are merely illustrative.

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Todos os direitos reservados | All rights reserved