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## User Guide

[www.wtavet.com.br](http://www.wtavet.com.br)

### **Aquecedor de Filtros - WTA**

### **Filter Warmer - WTA**

## ADVERTÊNCIA

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## Index

<b>WARMING</b> .....	12
Copyright .....	12
<b>PRESENTATION</b> .....	12
<b>ACCESSORIES</b> .....	13
<b>PRODUCT APPLICATION</b> .....	13
<b>TECHNICAL FEATURES</b> .....	14
<b>FUNCTION SPECIFICATION</b> .....	14
<b>IMPORTANT PRECAUTIONS</b> .....	15
Repairs .....	15
Cares and Transport .....	15
Cleaning .....	15
Falling Objects Inside the Equipment .....	15
Power Cable.....	15
Rain and Thunderstorms.....	15
<b>INSTALATION AND SETTING</b> .....	16
Connecting the Equipment.....	16
Inserting the Filter .....	16
Setting the Temperature .....	16
<b>LIMITED WARRANTY</b> .....	17
<b>PROBLEM SOLUTION</b> .....	19
Malfunctioning on the Power Grid.....	19
Heating Unit .....	19
<b>TECHNICAL ASSISTANCE SERVICE/CUSTOMER SERVICE</b> .....	19

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## USER GUIDE

### WARNING

To prevent accidents and/or damage, read and follow carefully this instruction guide before using the product.

Warning for dangerous voltage presence.

Warning for important operating instructions.

### Copyright

WTA – Filter Warmer - WTA user guide

WTA, Cravinhos, State of São Paulo, Brazil.

The content of this guide is protected by copyright laws and cannot be reproduced, distributed, disclosed neither stored without prior written permission of WTA - Watanabe Tecnologia Aplicada Eireli - EPP.

In order to obtain authorization, additional information or to clarify doubts, please contact our Technical Assistance Department by e-mail, letter or phone.

Phone: +1(979) 324 - 6168

E-mail: [usa1@wtatech.com](mailto:usa1@wtatech.com)

\*Opening hours: From Monday to Friday, 8:30 am to 5:30 pm (GMT -03:00)

©WTA - Watanabe Tecnologia Aplicada Eireli - EPP - All rights reserved.

### Comments

The information, illustrations and specifications contained in this user guide assume the latest features of the product available at the time of its publication.

Due to the use of new technologies and raw materials, WTA reserves the right to change the design and/or technical specifications of the product always as necessary and without prior notice, without incurring in any obligations.

This manual is an integral part of the product. Therefore, the consumer should save it for any queries.

## PRESENTATION

Congratulations on your choice. You have just purchased a WTA product, a brand that adds quality and technology in what it manufactures.


WTA is a Brazilian company specialized in solutions for animal reproduction. Our products are developed for working with cattle, sheep and horses, with recognized approval by the most renowned specialists in the world.

Its anodized aluminum frame offers, besides an elegant design, conditions that facilitate its internal and external cleaning.


For the best performance of the product we recommend reading this manual.

## ACCESSORIES

When acquiring your equipment, be sure that these accessories are intact and included in the package: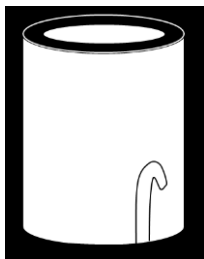


Device Warmer

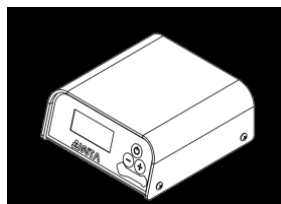Control Module

AC Power Supply  
Adapter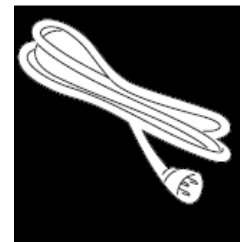

Power Cable

**Note:** If accessory is missing or defective, please contact WTA Technical Assistance, within 3 days counted from the product delivery date.

## PRODUCT APPLICATION




The WTA Filter Warmer is used to maintain a constant temperature during oocyte and embryo recovery or collection procedures. Practical and easy to use, the equipment has an electronic sensor that maintains the temperature during the work, with tolerance variation of  $\pm 0.5$  ° C. These features provide rapid and thermal safety during procedures

## TECHNICAL FEATURES

- Equipment manufactured in aluminum.
- Device warmer dimensions:
  - Height 153 mm / Hole 102 mm (Ø).
- Module size (W x L x H): 105 x 93 x 80mm.
- Maximum power: 64,5W.
- AC power supply: 100 ~ 240Vac - 50/60 Hz - 1,4A.
- DC unit supply: 15Vdc - 4,3A.
- Temperature adjustment: from 68.0 to 113.0°F or (20.0 to 45.0°C).
- Temperature range: ±1°F.
- Product limited warranty: 3 years.

## FUNCTION SPECIFICATION

- 01 - AC power supply
- 02 - Output DC
- 03 – Device warmer hole
- 04 – Drain opening
- 05 - Control module
- 06 - LCD display
- 07 - “ON/OFF” switch
- 08 - “+/-” Switch
- 09 – Power Input
- 10 – Device connection cable

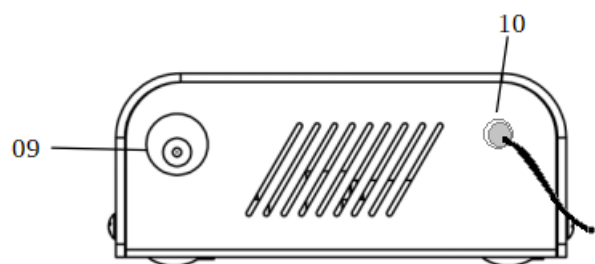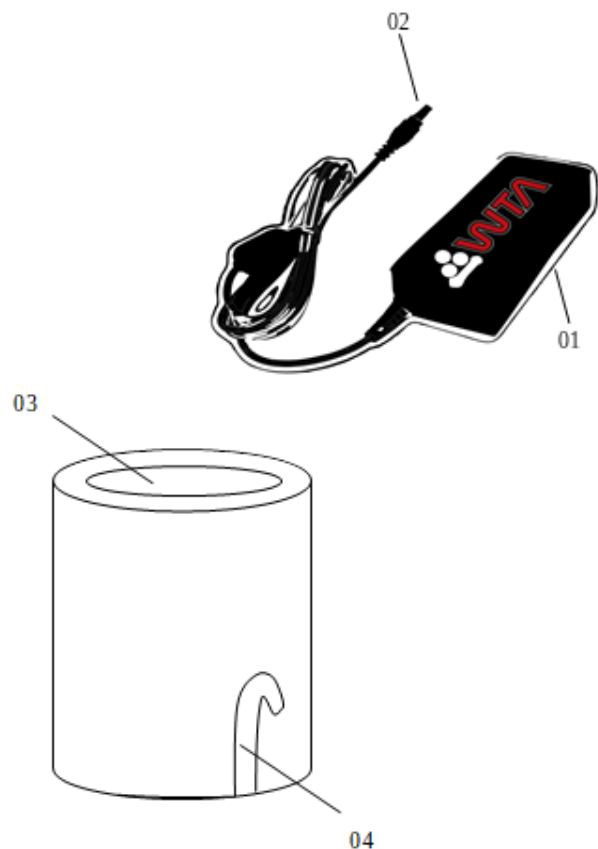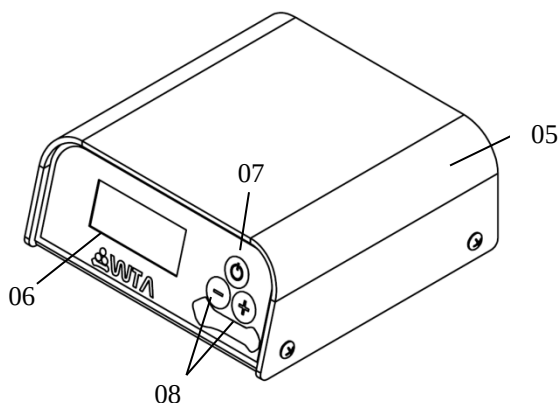## IMPORTANT PRECAUTIONS

### Repairs:

Do not remove the screws to open the equipment. There are no pieces in the equipment that can be fixed by the user. All repairs and maintenance should be performed by authorized professionals or the WTA Technical Assistance Service.

### Cares and Transport:

Do not transport the unit on places with much trepidation. Such procedure may damage the equipment. Always, make sure that while transporting the product, is well packed and protected against accidents shocks and impacts.

### Cleaning:

Clean the device with a soft cloth, slightly moistened with water. Do not use thinner or other strong solvents, because they may damage the device's finishing.

### Falling Objects Inside the Equipment:

Do not place glasses or cups containing liquid, coins, paper, clips or any other metal objects on the product. If you notice any liquid or objects felt inside the equipment, unplug it immediately and contact WTA's Technical Assistance.

### Power Cable:

Do not step on the power cable or put any kind of weight (furnishing, packages) over it.

### Rain and Thunderstorms:

In the event of thunderstorms with lightning, turn off (unplug) the equipment.

## INSTALATION AND SETTING

### Connecting the Equipment

For your safety, first connect the power supply (02) to the control module (09), then connect the power supply (01) to the power 100~240Vac.

### Inserting the Filter

Insert the filter through the opening (03). Adjust the drain duct filter so that it fits into the drain opening (04).

### Setting the Temperature

When connect the module the network power a beep will sounds. Turn the unit on by pressing the ON/OFF button (07) for 3 seconds, a beep will sound and the display (06) will display the device name, serial number and version. After another beep and while the display is displaying SETPOINT, adjust the desired temperature by pressing the "+" or "-" (08) buttons. It can be adjusted in a range from 68.0 to 113.0°F (20.0 to 45.0° C).

To change the temperature value turn the module off and repeat the previous steps.

This equipment will not provide cooling if temperature setted was lower than ambient temperature.

To turn the unit off, press and hold the ON/OFF button (07) for 3 seconds until the module beeps.

**Note:** If you are not going to use the product for a long time, unplug the power grid.

## LIMITED WARRANTY

WTA - Watanabe Applied Technology Eireli - EPP follows strict internal quality control standards in the products manufacture. In addition to the legal guarantee of 90 (ninety) days, provided for in the Consumer Protection Code, it guarantees its customers an additional 33-month Warranty, counted from the date of issue of the purchase invoice, provided that the equipment has been installed and according to the guidelines contained in the User's Manual.

Even if ownership of the equipment is transferred, the warranty will continue to be valid according to the terms and conditions specified above.

This product has been carefully evaluated at every stage of the manufacturing process. This warranty will always accompany you, valid only in the country where the product was purchased.

It is understood by Additional Warranty, the free repair of the product and / or the replacement of parts that, according to technical opinion of the WTA, present a manufacturing defect.

Only WTA Technical Assistance professionals are entitled to repair defects for the guarantee, by presenting the sales tax (original) to the first buyer / consumer.

### **The Warranty Service**

- Assistance in WTA technical assistance.
- Parts used for maintenance, subject to the exceptions provided in the item "Exclusion of Warranty".

The product should be sent directly to the WTA observing that:

- The product must be properly packed, if possible in its original packaging, together with the manual, cables and other accessories that have been delivered at the time of purchase.
- The product will be pre-evaluated by WTA technicians prior to repair.
- Shipping, freight and packaging costs will only be charged to WTA during the legal guarantee period (90 days). After this period such costs will be the customer's responsibility.

## **Voiding the Warranty**

This warranty does not cover defects or defects arising from the following situations:

- Installation and use of the product in an improper way or in disagreement with the respective guidelines and information described in this Manual, including installation, external adjustment, cleaning and use of accessories not in accordance with technical specifications.
- Defects caused by accidents, falls, atmospheric phenomena (rains, floods, electric discharges), excessive humidity (water, liquids in general), salinity, sinister (theft or theft), loss or improper transport of the product.
- Installation, adjustment, repair or servicing by persons outside WTA service.
- Invoice and / or serial number adulterated or withdrawn.
- Modifications, alterations or violation of the original product.
- Product connected to the mains improper or subject to excessive fluctuations.
- The product is purchased in balance, showcase or demonstration, this being characterized by means of identification in the invoice of purchase and / or price lower than the one practiced in the market.

**Important:** WTA is not liable for damage to equipment during transportation. When purchasing your product make sure on the delivery act that it is not defective or with apparent defects. If you notice any irregularities, return the product immediately.

## **This Warranty Statement Does Not Include**

- Periodic reviews.
- Displacement and expense of technician by authorized WTA service.

## **Responsibility**

The WTA has no control or influence over the conditions under which the equipment is used, as well as the methodologies employed. Thus, WTA assumes no responsibility for the results, use and / or performance of the equipment (including: damages for loss of profits, interruption of business activity, loss of business information or other pecuniary losses). WTA expects that the equipment will be used by trained personnel. The purchaser

acknowledges that the equipment is subject to all terms and conditions described in this manual.

## **PROBLEM SOLUTION**

Often what at first appears to be defective may be something simple to be solved. If you notice any problem related to the equipment operation, check the procedures listed below.

### **Malfunctioning on the Power Grid**

- Make sure the Power Cable is properly connected.
- Disconnect it from the power, then wait 5 seconds and connect it again.

### **Heating Unit**

- Make sure the temperature is set correctly.
- Make sure that the cable is properly connected.
- Disconnect it from the power. Then, wait 5 seconds and connect it again.

If the problem persists, please contact our Technical Assistance Service by making a detailed description of the problem.

## **TECHNICAL ASSISTANCE SERVICE/CUSTOMER SERVICE**

If there is any doubt or if it is necessary additional information about warranty, assembly, installation, adjustment or utilization of the product, please contact our Customer Service Department:

Opening hours: From Monday to Friday, 8:30 am to 5:30 pm (GMT -03:00)

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Images contained in the Guide are merely illustrative.

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Todos os direitos reservados | All rights reserved